

NEOGEN CORPORATION

944 NANDINO BLVD, LEXINGTON, KY, 40511-1205

Telephone: 859-254-1221

Order Desk: 800-525-2022

Fax: 859-255-5532

Website: www.neogen.com

Email: inform@neogen.com

CHONDROPROTEC®

Neogen

Polysulfated Glycosaminoglycan

Indications for Use:

Partial and Full Thickness Wounds, Secreting and Bleeding Dermal Lesions, 1st and 2nd Degree Burns, Pressure Ulcers (Stages I-IV), Surgical Incisions, Donor Sites, Venous Stasis Ulcers.

Caution: Federal Law restricts this device to sale by or on the order of a licensed veterinarian.

Directions for Use:

1. Irrigate wound site with sterile water or normal saline.
2. Apply CHONDROPROTEC liberally to wound surface.
3. Cover with gauze or a non-stick (Telfa) pad.
4. Redress wound site every 24-48 hours.

Store at controlled room temperature (15°-30°C).

Protect from freezing. Discard unused portion.

For use on dogs, cats and horses. Not for food animal use.

Sterile when unopened

Sold to Licensed Veterinarians Only

Distributed by: Neogen Corporation, 944 Nandino Drive, Lexington, KY 40511

Phone 800/525-2022 USA/CANADA Fax 859/255-5532

10 mL/ 1000 mg	L145-1097 Rev 0404
----------------	--------------------

NAC No.: 14910212